SpermFreeze Box

Document ID: FP09 1122 R01 A.1
Update: 09/01/2023

**MATERIAL INCLUDED/
MATÉRIEL INCLUS/
IM LIEFERUMFANG ENTHALTENE MATERIALIEN/
MATERIAL INCLUIDO/
MATERIALI INCLUSI/
MATERIAL INCLUSI/
MATERIAL INCLUIDO/
ΥΛΙΚΟ ΠΟΥ ΠΕΡΙΛΑΜΒΑΝΕΤΑΙ/
MATERIAAL AANWEZIG**

Catalogue number
Référence catalogue
Beestelnummer
Número de catálogo
Numero di catalogo
Αριθμός καταλόγου
Catalogusnummer

SPF_BOX SpermFreeze Box

CUSTOMER-TECHNICAL SUPPORT
SUPPORT CLIENTS-SUPPORT TECHNIQUE
KUNDENDIENST-TECHNISCHER SUPPORT
ATENCIÓN AL CLIENTE-ASISTENCIA TÉCNICA
ASSISTENZA CLIENTI-SUPPORTO TECNICO
APOIO AO CLIENTE-TÉCNICO
ΥΠΟΣΤΗΡΙΞΗ ΠΕΛΑΤΩΝ-ΤΕΧΝΙΚΗ ΥΠΟΣΤΗΡΙΞΗ
KLANTEN-TECHNISCHE ONDERSTEUNING

FertiPro NV
Industriepark Noord 32
8730 Beernem / Belgium
Tel +32 (0)50 79 18 05
Fax +32 (0)50 79 17 99
URL: www.fertipro.com
E-mail: info@fertipro.com

SpermFreeze Box

INTENDED USE

The SpermFreeze Box is a box to standardize your sperm freezing procedure. It offers a standardized height of your sperm freezing straw in the liquid nitrogen vapor phase.

For professional use only.

COMPOSITION

Box and floater in expanded poly propylene (EPP)

MATERIAL NOT INCLUDED BUT NEEDED

- Sperm freezing medium: SpermFreeze / SpermFreeze SSP (FertiPro NV)
- Sperm freezing straws
- Freezing tank with liquid nitrogen

METHOD

See figure 1:

- 1 Fill the box with 1-2 liter of liquid nitrogen
- 2 Place your sealed straws (maximum 15) on the floater
- 3 Deposit the floater on the liquid nitrogen
- 4 Close the box by placing the lid loosely on the box (do not close the lid too tightly, the importance is not to disturb the vapour phase)
- 5 Leave for (at least) 15 minutes
- 6 Plunge the straws in the liquid nitrogen
- 7 Store them in your nitrogen storage container for long term preservation

PRE-USE CHECKS

Before each use, check if the SpermFreeze Box shows any damage. Do not use if defects are noticed.

STORAGE INSTRUCTIONS

- Empty the SpermFreeze Box after each use
- The SpermFreeze Box can be used multiple times, is washable (dishwasher) and disinfectable (using disinfectants standardly used in an ART-lab)

WARNING AND PRECAUTIONS

Wear personal protective equipment during use and manipulations of liquid nitrogen to protect yourself

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